




USER GUIDE

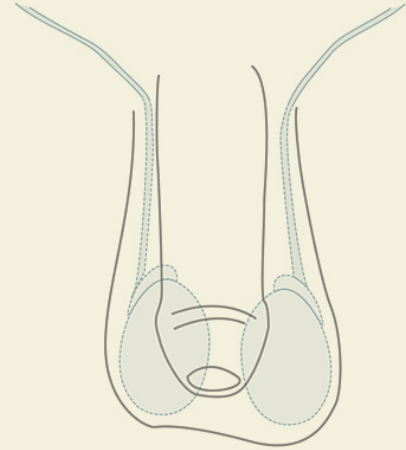

CLINICAL MALE PELVIC TRAINER (CMPT) MK 2

STANDARD

This product is available
in light and dark skin tones

60950

60964

FOR MORE SKILLS TRAINING PRODUCTS VISIT

limbsandthings.com

Sussex Street, St Philips, Bristol, BS2 0RA, UK | sales@limbsandthings.com | +44 (0) 117 311 0500

INDEX

PACKAGE SUPPLIED	3
SAFETY & CLEANING INSTRUCTIONS	4
SET UP	5
• SETTING UP THE CMPT	5
• ATTACHING THE FORESKIN	6
TRAINING	7
• STANDING & SUPINE	7
CLEANING & STORAGE	7
• CLEANING & STORAGE AFTER USE	7

TAKE A LOOK
AT OUR **VIDEO**



VISIT THE WEBSITE
limbsandthings.com

OR YOUTUBE CHANNEL
youtube.com/limbsandthings

PACKAGE
SUPPLIED

CMPT Abdominal Insert

60959

60973



Lower Torso



Base



CMPT Module 1: Normal

60952

60966



CMPT Foreskins (x2)

60961

60974



CMPT Pubic Bone

60960



Carry Case



Lined area for taking notes.

SAFETY & CLEANING INSTRUCTIONS

READ THE SAFETY INSTRUCTIONS CAREFULLY BEFORE USING YOUR CLINICAL MALE PELVIC TRAINER

DO's



Follow all instructions. Keep these instructions. Heed all warnings.

Only use water-based lubricant sachets, such as the ones supplied with this model.



Wear gloves at all times and remove all jewellery items. Sharp surfaces such as finger nails and rings can lead to damage of silicone parts.

Clean the product with a damp soft cloth or sponge, using only warm water with mild detergent, after every training session.



DON'Ts



Do not install near any heat sources, do not expose the product to flames or use near naked flame sources. Do not use flammable solvents near or on this product.

Do not use any attachments other than those supplied by the manufacturer.



Do not wear any jewellery items.

Do not use if the skin is torn.



Do not disassemble the product or attempt to repair a faulty unit, please return the product to Limbs & Things for assessment and repair.

STORAGE & TRANSPORTATION



Always ensure that the products are properly packed and secured during transportation and storage in order to prevent personal injury or damage to the product.



SET UP

SETTING UP THE CMPT


1 Removing/Attaching CMPT Module to Pubic Bone



CMPT Module - Rear view

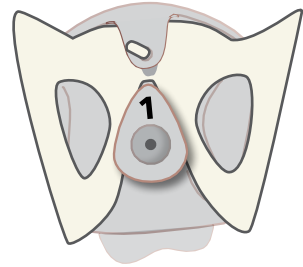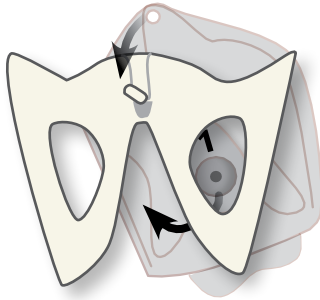

2 Fitting CMPT Module and Pubic Bone assembly in the Lower Torso


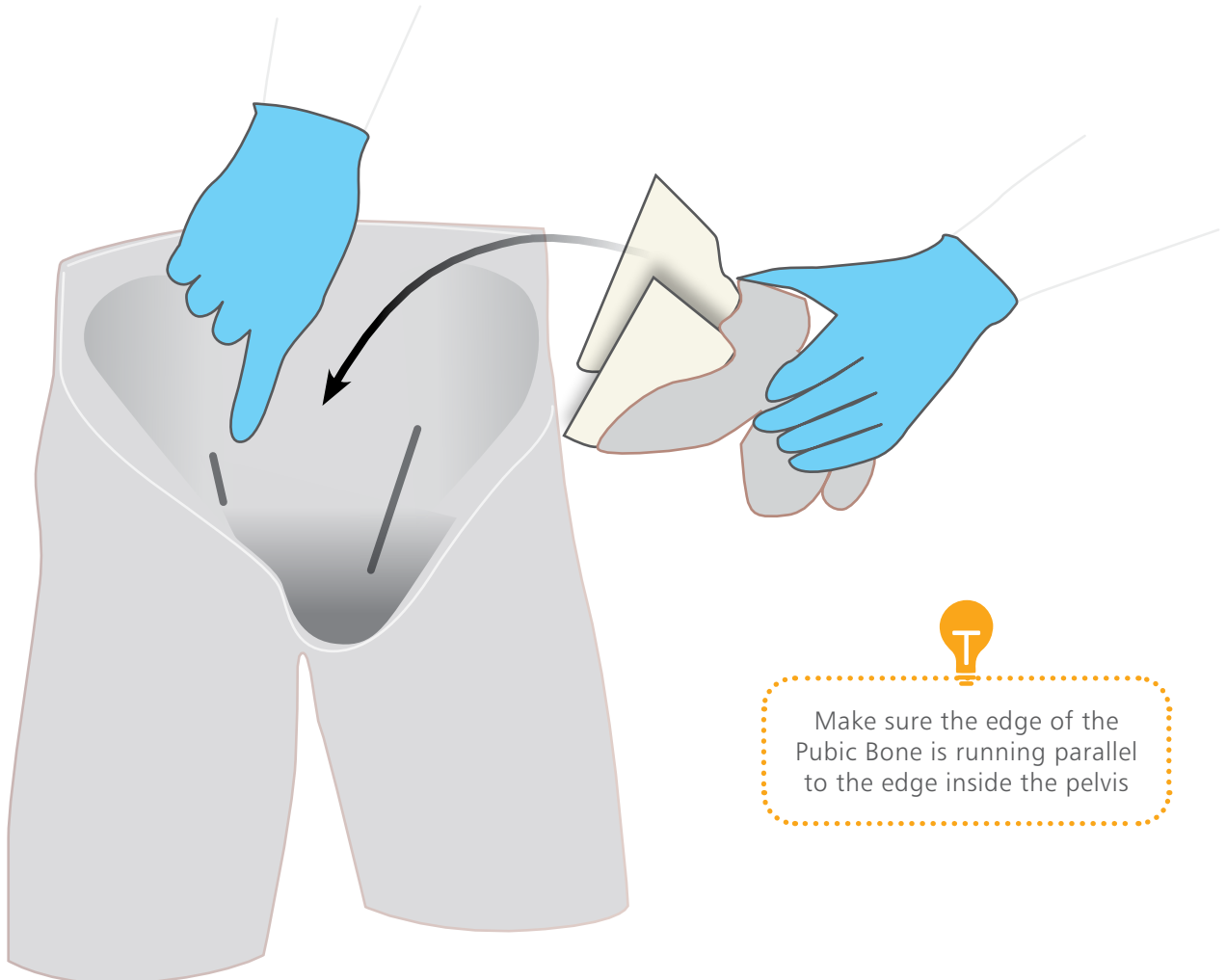


Make sure the edge of the Pubic Bone is running parallel to the edge inside the pelvis




SET UP

SETTING UP THE CMPT

3 Removing/Attaching Abdominal Insert

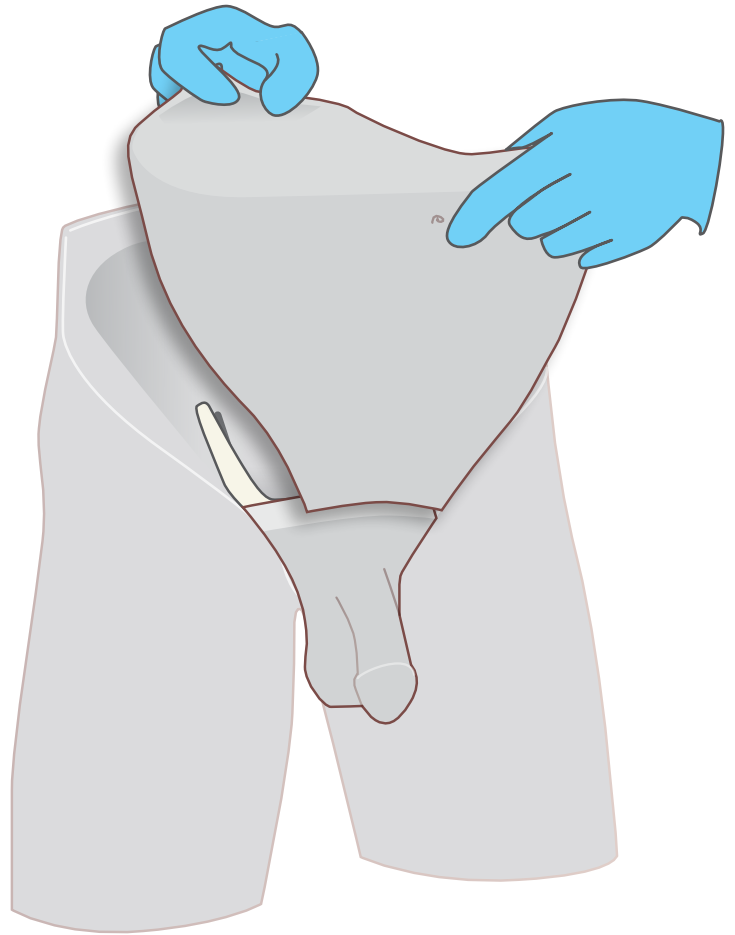
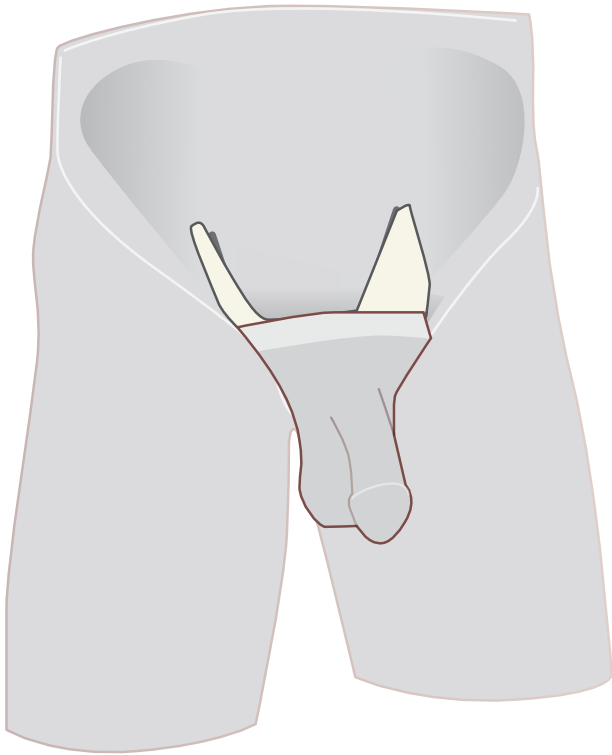


ATTACHING THE FORESKIN

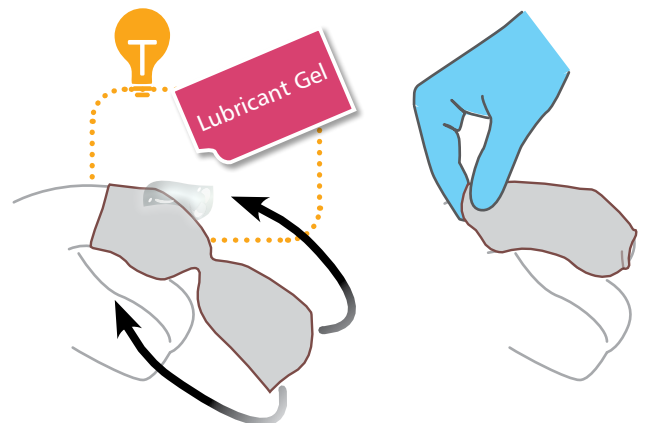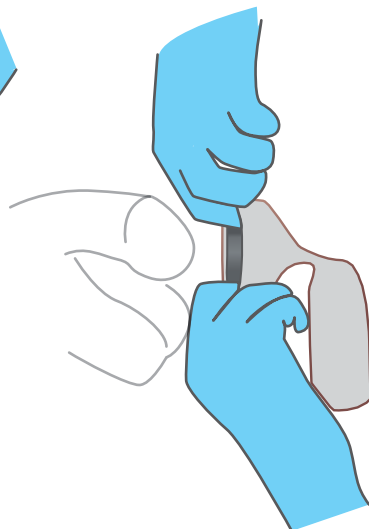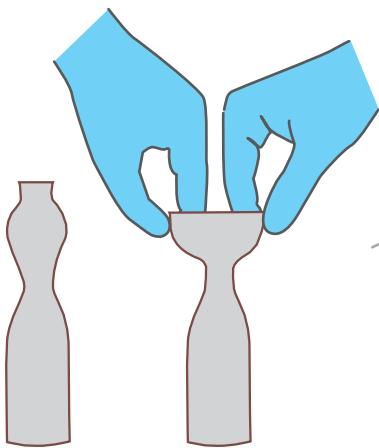


TRAINING

Can be used in 2 positions: standing & supine

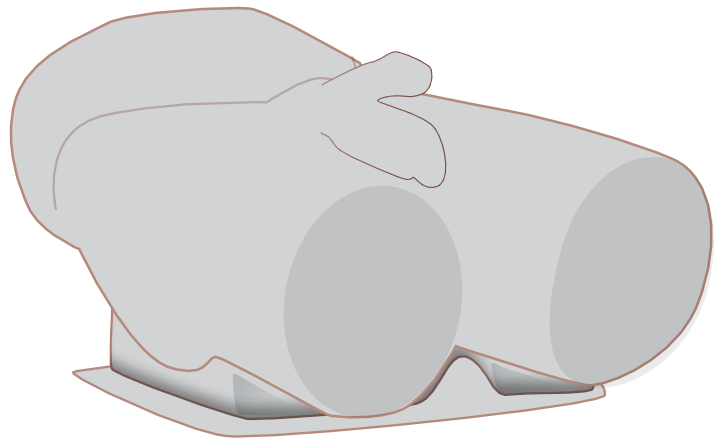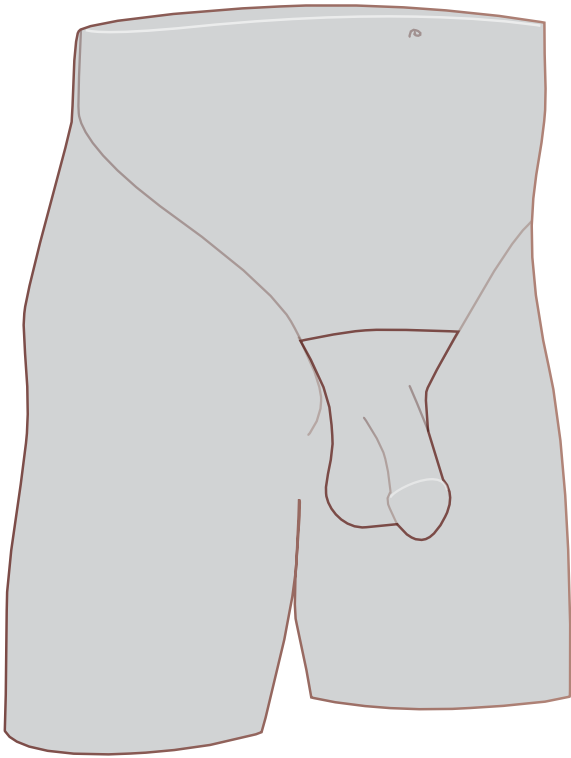

CLEANING
& STORAGE

Clean off any surface dirt with a damp, soft cloth, using only warm water and mild detergent

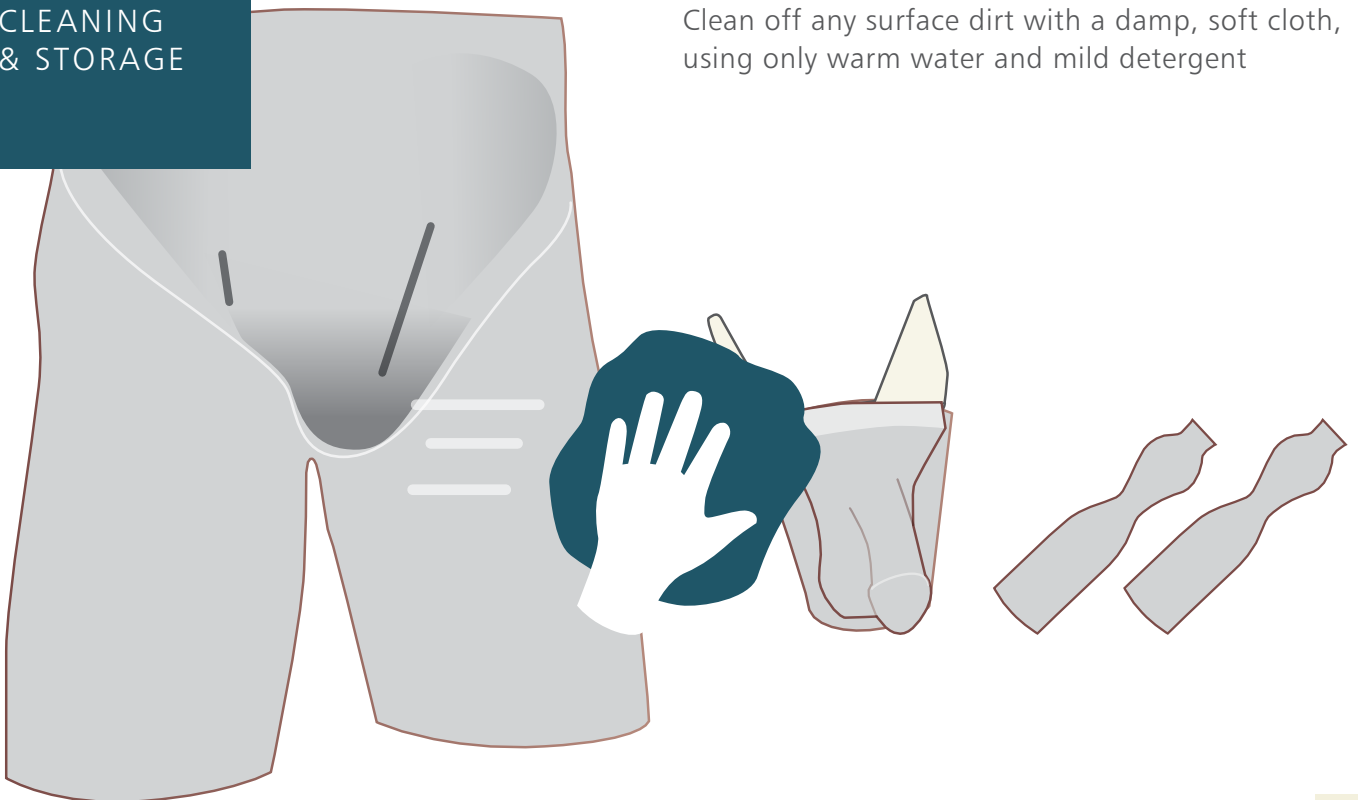

YOU MAY
ALSO BE
INTERESTED IN...


ADDITIONAL MODULES

CMPT MODULE 2

Varicocele

60953

60967

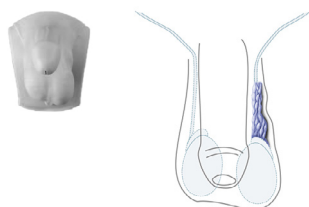


CMPT MODULE 3

Tumors

60954

60968

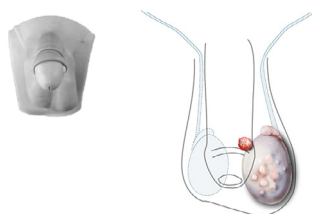


CMPT MODULE 4

Epididymal Cysts

60955

60969

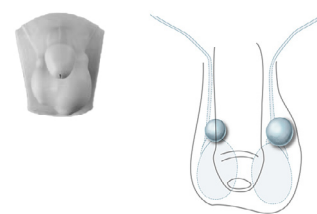


CMPT MODULE 5

Hydrocele

60956

60970

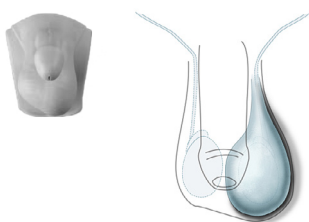


CMPT MODULE 6

Epididymitis/Epididymo-orchitis

60957

60971

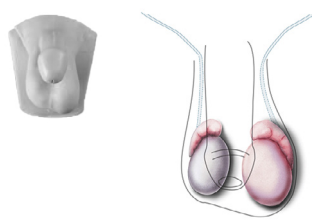


CMPT MODULE 7

Indirect Inguinal Hernia

60958

60972

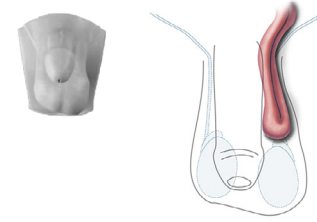

► Images above are for information purposes & show a simplified representation of anatomy within the CMPT Modules